***Clinical Assurance Program CAP
(Reject Analysis)***

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Klicka på Add

Avsluta med OK

Clinical Assurance Configuration

List of rejection reasons

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List of rejecting persons

- Andrea
- Barbara
- Frank
- Patrick
- Susanne

Patient moved

Add Remove Add Remove

Send rejected images to network node

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Abdomen, KUB

Abdomen

PT Position / Laterality:
Not Defined
Not Defined

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SIEMENS

The screenshot displays the Siemens Radiography Imaging System interface. The main window shows a KUB (Knee, Urinary Bladder, and Bowel) X-ray of the abdomen. A dialog box titled "Siemens Radiography Imaging System" is overlaid on the image, asking for confirmation to delete. The dialog box contains the text "Delete Confirmation. Are you sure?" and two buttons: "Yes" and "No". The "Yes" button is highlighted with a red box and a red number '2' in the top-left corner. The interface also includes a menu bar (Patient, View, Transfer, Options, Help), a toolbar with various icons for PT Position, View, Zoom/Pan/Rotate/Flip, Label/Comment, Filter, and Windowing, and a status bar at the bottom.

Abdomen, KUB

Abdomen

PT Position / Laterality:
Not Defined

View:
Zoom / Pan / Rotate / Flip:
Label / Comment:
Filter:
Windowing:
W: 2462
C: 1648

Siemens Radiography Imaging System

Delete Confirmation.
Are you sure ?

Yes No

LAT: N/A-PP
Abdomen
Ext: 344
Scale: 53%
kV: 75.0
mAs: 40.25

DV: 18
W:2462
C:1648

Not Ready - System in review mode

Import from off-line request for C:\SYNGO\TEMPICDR_OFFLINE.suc

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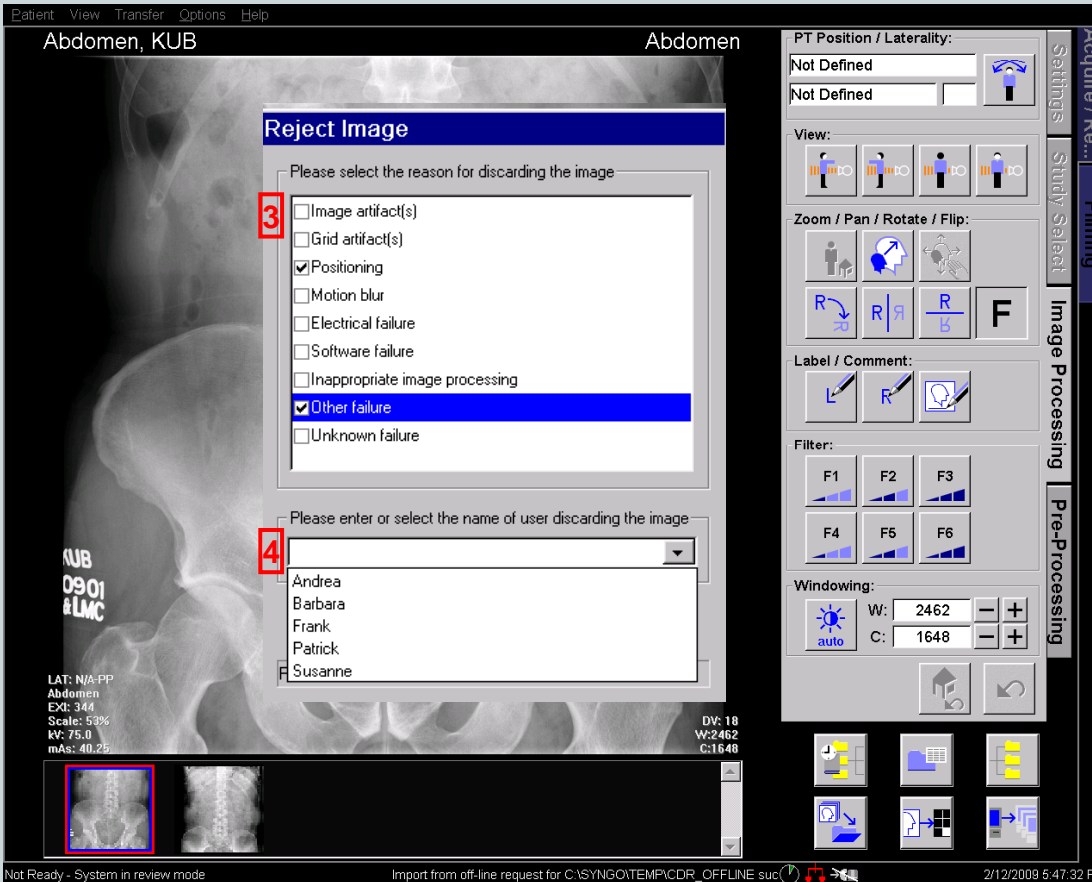
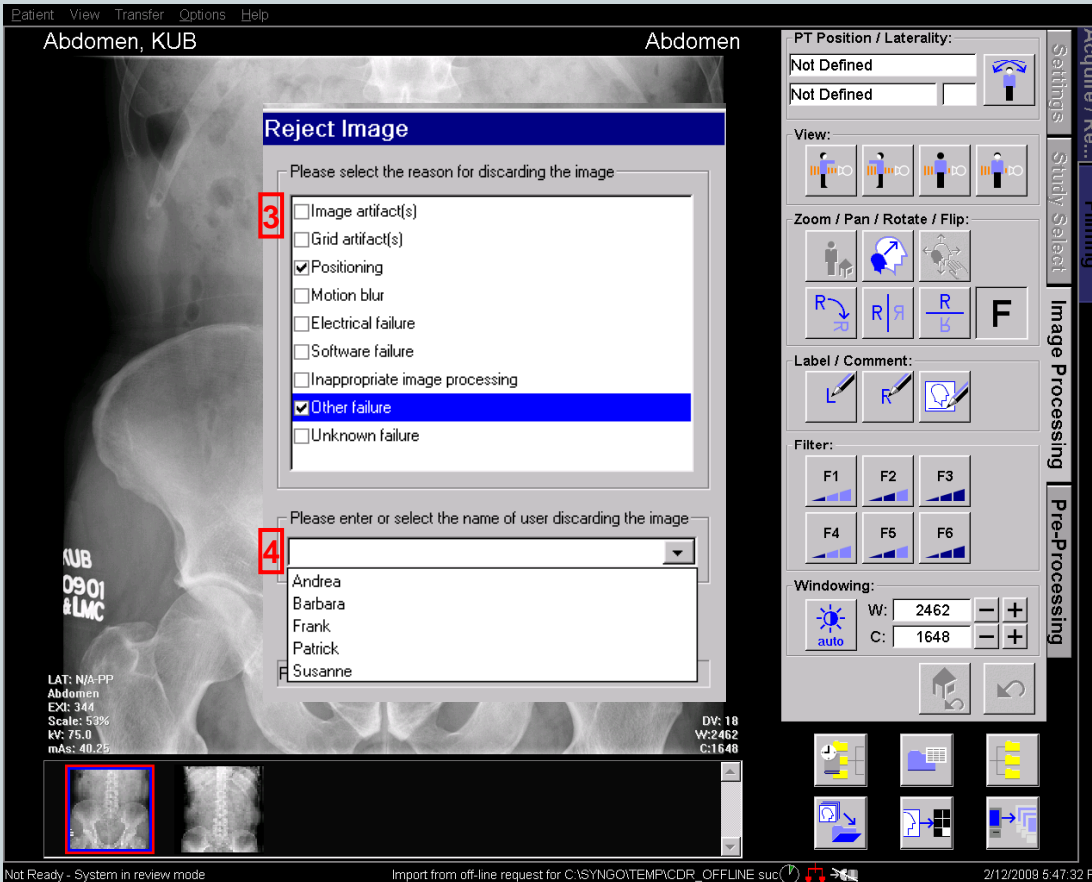
CAP

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Reject Image

Please select the reason for discarding the image

- Image artifact(s)
- Grid artifact(s)
- Positioning
- Motion blur
- Electrical failure
- Software failure
- Inappropriate image processing
- Other failure
- Unknown failure

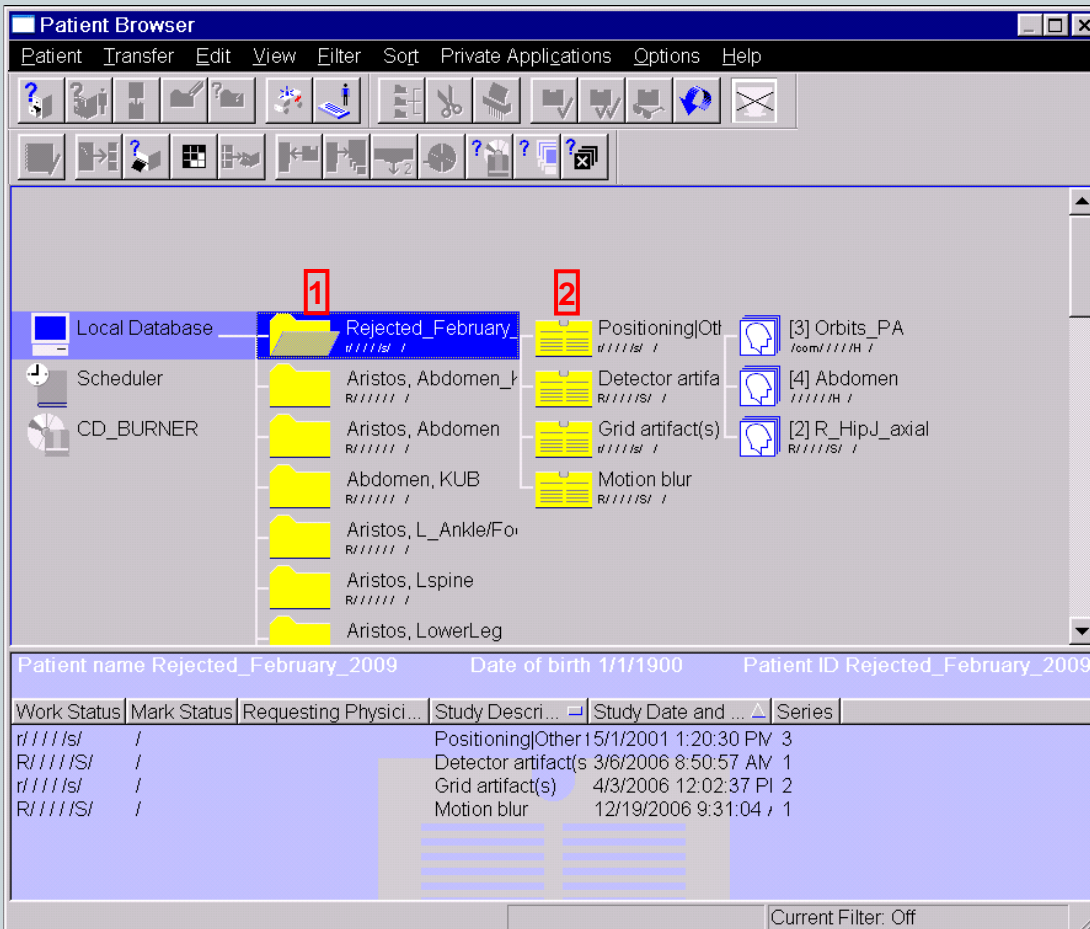
Please enter or select the name of user discarding the image

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Raderade bilder kan lagras i:

- modalitet: **Recycle bin function**
- PACS
- *syngo X-Workplace/ syngo MultiModality Workplace*

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Buttons: Delete All, Delete Until..., Save As..., Show Statistics..., Close

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Clinical Assurance Program (CAP) Statistik

Statistisk analysis

Management of rejected image data

Number of discarded images: 7

Rejection Date/Time	Acquisition Date/Time
02/13/09 16:06:00	05/09/01 10:5
02/13/09 16:08:49	05/01/01 13:1
02/13/09 16:09:35	12/19/06 09:3
02/13/09 16:10:46	03/06/06 08:5
02/13/09 16:12:24	12/07/05 10:3
02/13/09 16:12:47	04/24/06 12:5
02/13/09 16:13:41	04/03/06 12:1

Delete All Delete U

Statistics of rejected images

7 images from a total amount of 0 acquired images were rejected in the period 02/13/09 - 02/13/09

Percentage of rejected images: 0%

Reject statistics by reason:

Percentage	Reason
33.33%	Positioning
22.22%	Other failure
22.22%	Grid artifact(s)
11.11%	Motion blur
11.11%	Detector artifact(s)

Reject statistics by user:

Barbara

Percentage	Reason
50.00%	Positioning
33.33%	Other failure
16.67%	Motion blur

Close

Number	Requested procedure ID
	807162727-1
	915897

Close

Utskrift

Statistics of rejected images

7 images from a total amount of 0 acquired images were rejected in the period 02/13/09 - 02/13/09

Percentage of rejected images: 0%

Reject statistics by reason:

Percentage	Reason
33.33%	Positioning
22.22%	Other failure
22.22%	Grid artifact(s)
11.11%	Motion blur
11.11%	Detector artifact(s)

Reject statistics by user:

Barbara

Percentage	Reason
50.00%	Positioning
33.33%	Other failure
16.67%	Motion blur

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Save As

Please select drive:
A:\

Please enter file name (extension .CSV will be added automatically):
RetakesFeb09

Save Done

Delete All Delete Until... Save As... Show Statistics... Close

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